Usage Tips

1. The Uni-Grip® 360 bitetab can rotate anywhere on the base. If the bitetab becomes separated from the base, it can be reinserted at point **A**. **G**
2. The bitetab can extend past the edge of the base for a wide range of positioning options. To capture more area past the apices, make sure tab is fully extended past base **H**. To capture more of the incisal edge, the bitetab can be adjusted up on base. **I**
3. For maximum patient comfort, always place sensor in the center of mouth.
4. Instruct the patient to "bite lightly" to hold sensor in place.
5. For additional assistance in the United States, call RINN Customer Service at 800-323-0970. A training video is also available at www.UniGrip.com.

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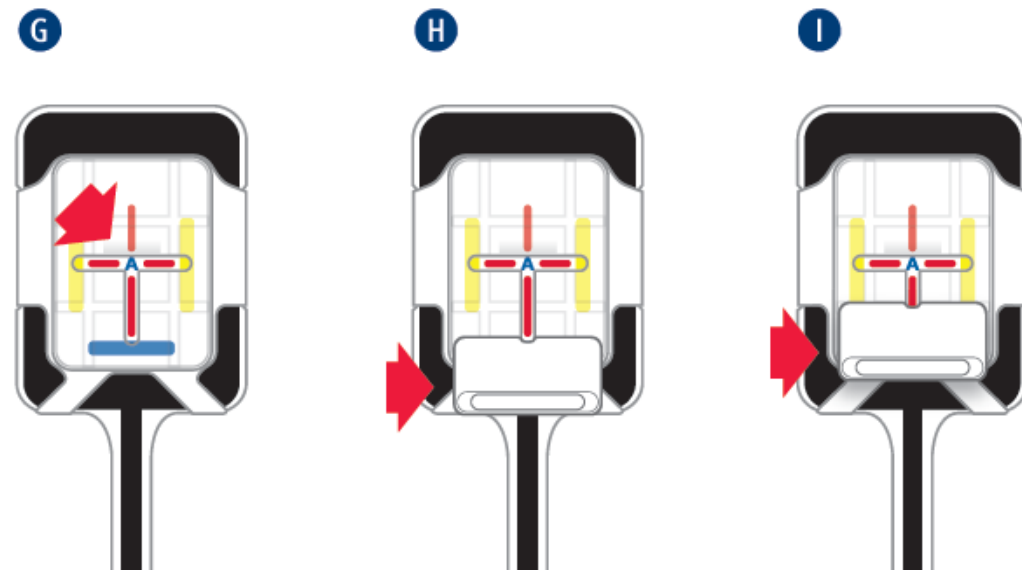
Manufactured in the USA by

DENTSPLY
RINN

A Division of
DENTSPLY International Inc.

1301 Smile Way
York, PA 17404 USA

www.DENTSPLY.com

Customer Service: 800-323-0970
Fax: 800-544-0787

EC REP DENTSPLY Limited

Building 3, The Heights,
Weybridge, Surrey KT13 0NY
United Kingdom

Rinn®, Uni-Grip® and Uni-GripAR®
are registered trademarks of
DENTSPLY International Inc.

Patented

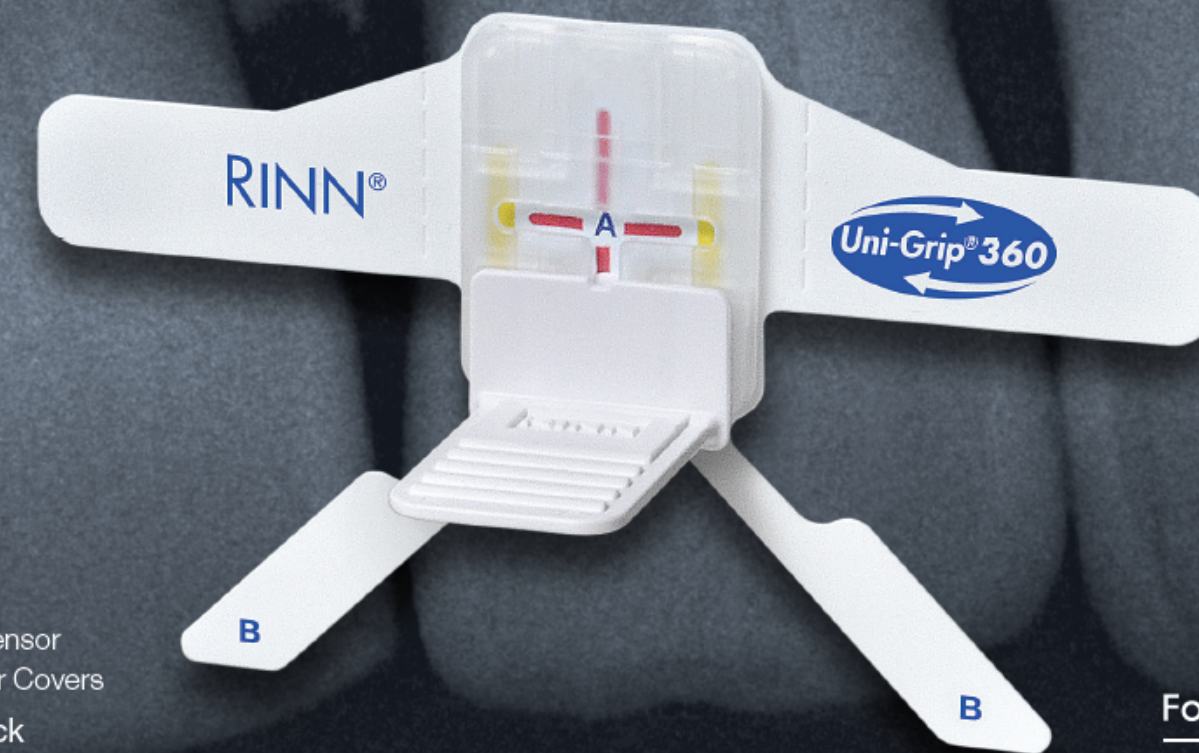
©2014 DENTSPLY International Inc.
All rights reserved.
Printed in the USA.

055916 Rev. 3 10.2014

UNI-GRIP® 360

UNIVERSAL SENSOR HOLDER

REF

550052 Uni-Grip® 360 50-Pack
50 Uni-Grip 360 Universal Sensor
Holders, 50 Universal Sensor Covers

550252 Uni-Grip® 360 Combo Pack
100 Uni-Grip 360 Universal Sensor
Holders, 100 Universal Sensor Covers,
1 Uni-GripAR Positioning Arm and Ring

550100 Uni-GripAR® Positioning Arm and Ring

550500 Rinn® Universal Sensor Covers,
package of 500

For better dentistry

DENTSPLY
RINN

Caution

U.S. Federal laws restrict this device to sale to or on the order of a licensed dental professional.

Indications for Use

Rinn Uni-Grip® 360 Universal Sensor Holder is indicated for use as a digital sensor holder during intraoral radiographs.

Contraindications

None known.

Warnings

Uni-Grip® 360 is single use only. Do not reuse.

Precautions

To be used only by individuals professionally trained to take intraoral radiographs. During use wear protective gloves. When using with digital sensor, only apply to protective sensor cover. Do not pull on sensor cord when removing. Only apply to dry surfaces.

Adverse Reactions

None known.

Storage

Store at room temperature (65-75°F/18-24°C), out of direct sunlight.

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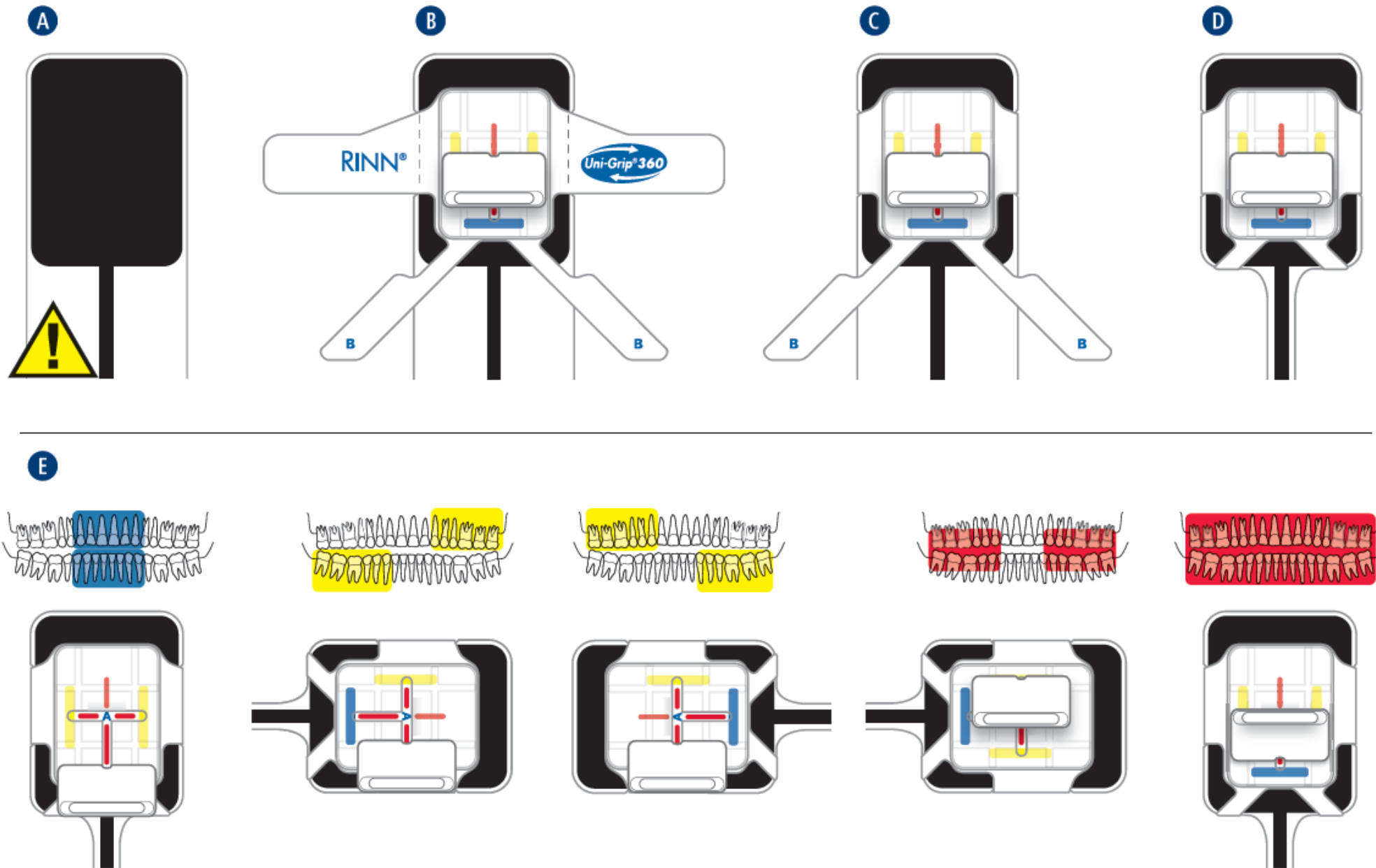
Step-By-Step Instructions

1. Cover digital sensor with sensor cover included in the Uni-Grip® 360 package. **A**
2. Remove backing liner from Uni-Grip® 360 and apply to center of active area with adhesive tabs marked **B** towards the bottom of the sensor where cord exits. **B**
3. Tightly wrap side tabs first **C** and then bottom tabs to back of sensor. **D**
4. Slide bite tab to desired position **E** and take x-ray. Repeat as necessary.
5. To remove, lift base to separate perforations on side tabs **F** and then remove bottom tabs. Do not pull on sensor cord when removing sensor from sensor cover.

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