Regisil®

DENTSPLY
CAULK

Vinyl Polysiloxane Bite Registration Material (*Tube Dispensed*)

DIRECTIONS FOR USE – ENGLISH

CLASSIFICATION

Vinyl Polysiloxane Bite Registration Material

INDICATIONS

Regisil® Bite Registration Material is designed for accurate interocclusal records of bite registration. This Regisil® Bite Registration Material offers minimum resistance to the patient's jaw closure. This minimizes deviation of the mandible and aids in producing consistently accurate centric relation records. This bite registration material accurately records occlusal details, enabling the dentist and lab technician to easily seat models with a slight amount of elasticity. This also aids in accurately seating models and dies by minimizing interferences from convex tooth surfaces.

Caution: U.S. Federal law restricts this device to sale by or on the order of a dentist. For dental use only.

COMPOSITION

Regisil® is a vinyl polysiloxane bite registration material.

Technical Data

Mix Time: 20-30 seconds

Work Time: 1 minute 30 seconds from the start of mix at 72°F (22°C)

Minimum Removal Time (MRT): 4 minutes from start of mix

CONTRAINDICATIONS

Do not use Regisil® Bite Registration Material as a temporary reliner.

PRECAUTIONS

1. Do not contaminate Regisil® Bite Registration Material with latex or "sulfur-containing" polymeric gloves. Residue from some brands of latex gloves may leach out especially in the presence of blood, saliva and some astringents and retraction cords. These residues on the surfaces of teeth can interfere with the chemical set of the polyvinyl surface and lead to loss of surface detail and inaccuracies. Vinyl or other non-latex gloves are recommended.
2. Use of some hand soaps and lotions can lead to interference with the setting reaction.
3. Before use, make certain the material is at room temperature.
4. Elevated temperatures will reduce work time and set times.
5. Optimal performance requires a base to catalyst ratio of 1:1. However, small variations in the relative amounts of base and catalyst will neither significantly increase nor decrease work or set times.

STEP BY STEP INSTRUCTIONS FOR USE

1. Dispense

Dispense equal lengthlengthss of Regisil® Bite Registration Material base and catalyst onto the mix pad provided for a base to catalyst ratio of 1:1.

2. Mix

In 20-30 seconds, using a stiff spatula, incorporate catalyst into base until mix is streak free.

3. Load Syringe and Use with or without tray

The mixed material is loaded into the supplied syringe using a small spatula. Filling the syringe may also be accomplished by scraping the material from the mix pad directly into the back end of the syringe barrel. This material may be used with or without a bite registration tray, but in most applications, a tray will not be required.

4. Insertion of Regisil® Bite Registration Material

The material is then injected directly onto the patient's teeth. The areas where teeth have been prepared should be injected first. Usually, only the teeth of the upper or lower arch need be covered. However, in recording the anterior teeth, it is suggested that material be placed over the teeth of both arches. A layer of about 5 mm of Regisil® Bite Registration Material is usually sufficient. In those situations where use of a tray is desired, the mixed material is placed on both sides of the tray to a depth of 2-3 mm. This material has been found to work well with mesh fabric type trays.

5. Removing Bite Registration Material

After 4 minutes from the start of mixing, the material may be removed from the patient's mouth. Have the patient open and carefully release the bite registration from the teeth. Abrupt force at this stage could cause permanent deformation of the bite registration.

6. Model Placement into Registration

Any excess material extending onto soft tissue areas or below the height of contour of the teeth should be removed to obtain maximum accuracy when seating the registration on the model. Excess material extending into interproximal areas can interfere with model placement and should be removed. Best results in trimming are obtained with a sharp scalpel blade or fine, sharp scissors.

7. Cleaning Instructions

Removal of unused hardened material is not difficult if plunger is left in material for about five minutes; then it will pull clean. Otherwise, the material will be very difficult to remove.

DISINFECTION

Regisil® Bite Registration can be disinfected with a standard disinfecting solution or spray. In an effort to control transmission of infectious diseases, the impression material may be treated using guidelines set forth in JADA, Vol. 116, February 1988, p.241.

STORAGE AND SHELF LIFE

Store and use Regisil® PB™ Registration Material between 65°-80°F (18°-25°C) and in relative humidity of 50%±20%. Do not use after expiry date.

©1998 DENTSPLY International Inc. All Rights Reserved
Printed in U.S.A.

CE Manufacturer:
DENTSPLY Caulk
Lakeview & Clark Avenues
Milford, DE 19963-0359, USA
Phone 1-800-532-2855

EU-Authorized Representative:
DENTSPLY DeTrey GmbH
De-Trey-Str. 1
D-78467 Konstanz
Tel. (0 75 31) 5 83-0

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