Aquasil Ultra

DENTSPLY

Smart Wetting® Impression Material

Aquasil Ultra XLV | Aquasil Ultra LV | Aquasil Ultra Monophase | Aquasil Ultra Heavy | Aquasil Ultra Rigid | Aquasil Ultra Xtra

DIRECTIONS FOR USE – ENGLISH

For dental use only.
U.S.A.: Rx only.

1. PRODUCT DESCRIPTION

Aquasil Ultra Smart Wetting® Impression Material is a quadrifunctional hydrophilic addition reaction silicone elastomeric dental impression material with excellent hydrophilic properties, dimensional accuracy, high tear strength, and resistance to permanent deformation. Aquasil Ultra Impression Material with Smart Wetting® improves both wetting on the tooth, tissue and abutment surfaces and model detail reproduction.

1.1 Delivery forms*

Aquasil Ultra Smart Wetting® Impression Material is available in:

- Light-bodied consistency, Medium-bodied consistency and Heavy-bodied consistency
- digit® Targeted Delivery System unit dose dispensed delivery, 50mL cartridge delivery and 5:1 DECA™ delivery for Dynamic Mixing Machines.
- Super Fast Set, Fast Set, Regular Set and Extended Time

*Some delivery forms may not be available in all countries

1.2 Composition

Polydimethylsiloxane polymer; Polymethylhydrogen Siloxane; Silicon Dioxide; Sodium Aluminosilicate; Organic Platinum Complex; Surfactant; Titanium Dioxide; Metallic Oxide Pigments; Peppermint Oil

1.3 Indications for use

Aquasil Ultra Smart Wetting® Impression Material is indicated for all dental impression techniques.

1.4 Contraindications

Aquasil Ultra Smart Wetting® Impression Material is contraindicated for use with patients who have a history of severe allergic reaction to peppermint oil or any of the components

1.5 Technical specifications

Classification: ISO 4823

Type 3: *Light-bodied consistency*: Aquasil Ultra XLV, Aquasil Ultra LV and Aquasil Ultra Xtra (wash)

Type 2: *Medium-bodied consistency*: Aquasil Ultra Monophase, Aquasil Ultra Heavy and Aquasil Ultra Xtra (tray)

Type 1: *Heavy-bodied consistency*: Aquasil Ultra Rigid

Working Time (22°C/72°F).....	Super Fast Set: 0'45"-1'15" (room temperature mixing and preparation, maximum insertion time)	Fast Set: 1'15"-1'45" Regular Set: 2'15"-2'45" Aquasil Ultra Xtra (wash): 4'30" Aquasil Ultra Xtra (tray): 2'30"-3'00"
Working Time (37°C/98°F).....	Super Fast Set: 0'35" maximum (intraoral syringing prior to tray insertion)	Fast Set: 0'35" maximum Regular Set: 1'10" maximum Aquasil Ultra Xtra (wash): 1'45" maximum
Setting Time (Mouth Removal Time).....	Super Fast Set: 2'30" from start of mix	Fast Set: 3'00" from start of mix Regular Set: 5'00" from start of mix Aquasil Ultra Xtra: 5'30" from start of mix
% Linear Dimensional Change.....	< 0.50	
% Recovery from deformation.....	≥ 98	
Detail Reproduction.....	.20 microns	

1.6 Compatible tray adhesive

Aquasil Ultra Smart Wetting® Impression Material is compatible with conventional tray adhesives designed for use with vinyl polysiloxane materials (See complete Directions for Use).

2. GENERAL SAFETY NOTES

Be aware of the following general safety notes and the special safety notes in other chapters of these directions.

Safety alert symbol

This is the safety alert symbol. It is used to alert you to potential personal injury hazards. Obey all safety messages that follow this symbol to avoid possible injury.

2.1 Warnings

Aquasil Ultra Smart Wetting® Impression Material contains polymerizable monomers and surfactants which may be irritating to skin, eyes and oral mucosa and may cause allergic contact dermatitis in susceptible persons.

- **Avoid eye contact** to prevent irritation and possible corneal damage. In case of contact with eyes, rinse immediately with plenty of water and seek medical attention.
- **Avoid skin contact** to prevent irritation and possible allergic response. In case of contact, reddish rashes may be seen on the skin. If contact with skin occurs immediately remove material with cotton and wash thoroughly with soap and water. In case of skin sensitization

or rash, discontinue use and seek medical attention.

- **Avoid ingestion/swallowing** of material to prevent irritation or obstruction. If accidental swallowing occurs, drink lots of water. This material is not hazardous when small quantities are ingested. Larger quantities may cause bowel obstruction. Seek medical attention in the event of digestive irregularities.
- **Do not use** Aquasil Ultra Smart Wetting® Impression Material as a temporary reliner. Use of the material in this manner may cause irritation to the oral mucosa. If symptoms occur discontinue use immediately. If irritation does not subside consult a physician.
- **Avoid multiple impressions** at one sitting (more than three of the same arch/area) to prevent irritation to the oral mucosa. If symptoms occur discontinue use immediately. If irritation does not subside consult a physician.

2.2 Precautions

- This product is intended to be used only as specifically outlined in the Directions For Use. Any use of this product inconsistent with the Directions For Use is at the discretion and sole responsibility of the practitioner.
- Insufficient data exist to support the use of Aquasil Ultra Smart Wetting® Impression Material for impressions on exposed bone. Aquasil Ultra Smart Wetting® Impression Material should only be used on or around intact mucosa.
- Wear suitable protective eyewear, mask, clothing and gloves. Protective eyewear is recommended for patients.
- Devices marked "single use" on the labeling are intended for single use only. Discard after use. Do not reuse in other patients in order to prevent cross-contamination.
- All products should be used at room temperature. Higher temperatures reduce work times and laboratory bench set times (faster), lower temperatures increase them (slower).
- To obtain optimum physical properties, the mix should be completely homogeneous (streak free).
- Aquasil Ultra Smart Wetting® Impression Material should be used with properly fitted, rigid trays for most applications. Trays with retentive features such as DENTSPLY Caulk Rim-Lock® Trays or perforations, pretreated with tray adhesive, are recommended. To reduce gagging, choking or swallowing, do not overfill tray.
- Material should extrude easily: DO NOT USE EXCESSIVE FORCE. Excessive pressure may result in unanticipated extrusion of the material or cause cartridge rupture.
- Safety and efficacy of Aquasil Ultra Heavy, Aquasil Ultra Rigid, Aquasil Ultra Xtra Tray material when used as an intraorally syringed (wash) material has not been demonstrated.
- Due to the stiffness of Aquasil Ultra Rigid Impression Material, severe undercuts should be blocked out prior to tray insertion. For full arch dentate impressions, select other Aquasil Ultra viscosities for tray material.

Interactions:

- Do not use Hydrogen peroxide-containing dental materials in conjunction with this product since they may interfere with the hardening of the polymeric components.
- Use of latex or "sulfur-containing" polymeric gloves, some hand soaps and lotions can lead to interference with setting reaction. Contamination may accumulate on the surface of the skin and may not easily wash off. Non-latex gloves worn during use will avoid this possibility.
- Handle cords with college pliers. Do not touch prepared tooth with gloves. Rinse preparation with copious amounts of water before making the impression.
- Aquasil Ultra Smart Wetting® Impression Material should not be intermixed with, or used in conjunction with, other manufacturer's vinyl polysiloxanes or any polyether, polysulfide or conventional (condensation cured) silicones.
- The dual phase, two-step ("putty/wash") crown and bridge impression technique is only recommended with Aquasil Ultra Xtra Tray Material Aquasil Putty (all versions) or Reprosil® impression materials (available separately) for primary tray impressions.

2.3 Adverse reactions

- **Eye contact:** irritation and possible corneal damage.
- **Skin contact:** irritation or possible allergic response. Reddish rashes may be seen on the skin.
- **Bowel obstruction** or other digestive distress may result from ingestion of mixed impression material. (See Warnings)

2.4 Storage conditions

Inadequate storage conditions may shorten the shelf life and may lead to malfunction of the product.

- Store at temperatures between 10°C/50°F–24°C/75°F.
- Use the product at room temperature.
- Do not store used cartridges with new (unused) mixing tips attached.
- Do not freeze.
- Protect from moisture.
- Do not use after expiration date.

3. STEP BY STEP INSTRUCTIONS

3.1 Cartridge dispensing gun

1. Insert cartridge into dispensing gun.
2. Bleed cartridge before installing new mixing tip
3. Preparation of syringe/wash material

- Dispense directly to the mouth with the intraoral tip; **OR**
- An intraoral syringe may be filled with mixed material from the cartridge; **OR**
- Prepare digit® Targeted Delivery System unit dose dispensed delivery wash/syringe material according to the Directions for Use.

NOTE: Do not inject Aquasil Ultra Heavy, Aquasil Ultra Rigid or Aquasil Ultra Xtra Tray Impression Material intraorally.

4. Disinfect cartridge dispensing gun, cartridge and attached used mixing tip for storage.
 - **Immediately prior to subsequent usage**, remove the used mixing tip, bleed cartridge, install a new mixing tip.
 - Dispose of cartridge when empty. Do not attempt to refill or consolidate used cartridges.

3.2 digit® Targeted Delivery System unit dose dispensed

1. Insert plunger into open end of syringe barrel.
2. Snap circular disk off of end of cartridge.
3. Insert unit dose cartridge into mixing tip. Ensure both cartridge spurs fully lock into mixing tip slots. Attach intraoral tip.
4. Insert assembled cartridge/mixing tip (with intraoral tip) into the syringe side opening. Slide mixing tip through barrel end with cross notches. Pull on mixing tip to ensure complete seating.
5. Dispense the material by applying slow, steady pressure on plunger. Do not use excessive force.
6. To disassemble, withdraw the plunger, tap mixing tip against the counter to dislodge, and remove through side opening.
7. Properly discard used cartridge/mixing tip and prepare syringe for reuse.

3.3 DECA™ cartridge loading

NOTE: Heavy cartridge – dropping may result in damage to the cartridge rendering it unusable. The DECA™ Cartridge fits Duomix, duomix II, Pentamix®, MixStar, Dynamix and Volume Mix dynamic mixers. Carefully remove dispensing cartridge from the package. Also refer to separate dynamic mixer manufacturer's instructions.

1. Remove the cap from the cartridge.
2. Insert the cartridge into the dynamic mixer.
3. Bleed the cartridge before installing a new mixing tip.
4. Install a mixing tip. Use only DENTSPLY mixing tips. Other tips may not seal properly.
5. Place locking ring over the mixing tip and turn until ring is locked in place.
6. Dispense the mixed base and catalyst into an intraoral syringe (Aquasil Ultra Monophase only) or a suitable tray.
 - **Immediately prior to subsequent usage**, remove the used mixing tip, bleed cartridge, install a new mixing tip.

3.4 Impression technique: Single or Dual Phase one-step ("double-mix") Technique with a Rigid Tray or with Double Arch Technique

NOTE: Tray and wash material mix timing – It is highly recommended to utilize an assistant to assure simultaneous syringing and tray loading.

1. Assure hemostasis and clean the field with air water spray.
2. *Syringe material:* Backfill the impression syringe or attach an intraoral tip to the end of the mixing tip, or prepare digit® Targeted Delivery System unit dose dispensed syringe
3. *Tray material:* Dispense material (assistant) into impression tray. Do not overfill tray.
3. Simultaneous with tray loading, syringe (dentist) completely around the preparation/aboutment.

NOTE: Tray and wash material insertion timing:

- *Super Fast Set wash and tray material:* Loaded tray must be inserted within 35 seconds from time of first syringing of wash material intraorally.
- *Fast Set wash and tray material:* Loaded tray must be inserted within 35 seconds from time of first syringing of wash material intraorally.
- *Regular Set wash and tray material:* Loaded tray must be inserted within 1 minute 10 seconds from time of first syringing of wash material intraorally.
- *Aquasil Ultra Xtra extended time wash and tray material:* Loaded tray must be inserted within 1 minute 45 seconds from time of first syringing of wash material intraorally.

4. Insert filled tray into the mouth.
5. Minimum Removal Time (from the start of mix)
 - *Super Fast Set:* 2 minutes 30 seconds
 - *Fast Set:* 3 minutes
 - *Regular Set:* 5 minutes
 - *Aquasil Ultra Xtra:* 5 minutes 30 seconds

NOTE: If Regular Set Wash is used with Fast Set Tray material, Minimum Removal Time is 4 minutes from start of mix.

6. Follow disinfection procedures before shipping and/or pouring.

Timing Guide	Super Fast Set: up to 0'35" (35 sec) Fast Set: up to 0'35" (35 sec) Regular Set: up to 1'10" (1 min 10 sec) Aquasil Ultra Xtra: up to 1'45" (1 min 45 sec)	Super Fast Set: 2'30" (2 min 30 sec) Fast Set: 3'00" (3 min) Regular Set: 5'00" (5 min) Aquasil Ultra Xtra: 5'30" (5 min 30 sec)
0'00" (0 sec)		

Begin simultaneous syringing and tray loading | Complete syringing and seat loaded tray | Remove set impression

3.5 Impression technique: Dual Phase two-step ("putty-wash") Technique with a primary tray impression

1. Complete primary impression with Aquasil Ultra Xtra Tray, Aquasil Putty or Reprosil® Tray material.
 - Place one or more polyethylene plastic sheets over the impression material surface. This sheet creates the uniform spacing required for this technique. It also protects the surface from contamination.
 - Insert filled tray into the mouth. Move tray slightly from side to side and back and forth. This creates the spacing for the wash material.
 - Remove tray from the mouth when material has reached a rubbery consistency and set aside. Do not disturb impression until fully set.
 - Remove polyethylene sheet, rinse impression with water and air dry.
2. It is recommended to utilize an assistant and two cartridge guns, or one cartridge gun and digit® Targeted Delivery System unit dose dispensed syringe to assure simultaneous syringing

and tray loading.

3. Follow impression steps above, using wash material for both intraoral syringing and loading into primary tray impression.

3.6 Impression Technique: Edentulous/Overdenture Impression Technique

Aquasil Ultra Monophase, Aquasil Ultra Heavy and Aquasil Ultra Rigid are acceptable for border molding.

1. Border Molding Technique
 - Dispense material on to the borders of the tray.
 - Insert the tray and perform conventional border molding techniques.
 - When material has reached a firm consistency, remove tray and examine borders. The addition of a small mix can be used if voids or deficient areas are present. Over-extensions can be easily removed with a scalpel blade.
2. The final impression can be made with Aquasil Ultra XLV or LV, Aquasil Ultra Monophase, Aquasil Ultra Heavy or Aquasil Ultra Xtra Material.
3. Insert filled tray into the mouth.
4. Observe the Minimum Removal Time of the selected Aquasil Ultra final impression material.
5. Follow disinfection procedures before shipping and/or pouring.

3.7 Impression technique: Duplication of Models

1. Load the impression tray with Aquasil Ultra tray material.
2. It is suggested to simultaneously syringe, using Aquasil Ultra wash material, completely around the model teeth.
3. Seat model into the loaded tray. Retain impression in position until firmly set.
4. Remove model from impression by pulling slowly to break seal.

4. HYGIENE

Cross-contamination

- Do not reuse single use products. Dispose in accordance with local regulations.
- Reprocess reusable products as described below.

4.1 Cleaning the dispenser

The cartridge dispensing guns and digit® Targeted Delivery System syringe may be cleaned by scrubbing with hot water and soap or detergent. Excess material may be cleaned with alcohol-moistened gauze. Refer to Dynamic Mixer manufacturer's instructions for dispenser cleaning and disinfection instructions.

4.2 Disinfection of the dispenser gun and cartridge and digit® Targeted Delivery System syringe

Disinfect dispenser guns, cartridges and used tips left in place on cartridges for storage and digit® Targeted Delivery System syringe with a hospital-level tuberculocidal disinfectant according to national and local regulations. Use impregnated wipes not sprays. **NOTE:** Vigorous wiping can destroy the label. Wipe cartridge gently. Some phenolic based agents and iodophor based products may cause surface staining.

Agents containing organic solvents, such as alcohol may tend to dissolve the plastic dispenser and plunger. The disinfectant manufacturer's directions should be followed properly for optimum results.

Sterilization

Following cleaning and disinfection, digit® Targeted Delivery System syringe may be steam autoclaved at 134°C, minimum 3 minute holding time. **Technique Tip:** Unset Aquasil Ultra Smart Wetting® Impression Material can be removed from clothing with dry cleaning solvent.

4.3 Disinfection of the impression material

The impression should be disinfected with a hospital-level tuberculocidal disinfectant according to national and local regulations.

- Iodophors, sodium hypochlorite (0.525%), chlorine dioxide, and dual or synergized quaternary ammoniums are approved disinfectants.
- Thoroughly soak by spraying or immerse the impression in any recommended hospital level disinfectant for the contact time recommended by the disinfectant manufacturer for optimum results.
- Water-based disinfectant solutions are preferred.

4.4 Casting

- The impression should be removed from the disinfectant, rinsed with water and lightly air dried.
- The impression may be poured in plaster or stone 30 minutes after disinfection or delayed up to fourteen days. For epoxy dies, delay pour 60 minutes.
- Do not store impression in direct sunlight.
- If the impression is to be shipped, use suitable packaging to preclude distortion.
- The material is compatible with a range of dental plasters and die stones. Impressions may be silver or copper-plated.

5. LOT NUMBER AND EXPIRATION DATE

1. Do not use after expiration date. ISO standard uses: "YYYY/MM."
2. The following numbers should be quoted in all correspondences:
 - Reorder number
 - Lot number
 - Expiration date

 Manufactured by:
DENTSPLY Caulk
38 West Clarke Avenue
Milford, DE 19963 USA
Tel: 1-302-422-4511
www.dentsply.com

 EC REP
DENTSPLY DeTrey GmbH
78467 Konstanz
Germany
Tel: 49-7531-583-0
www.dentsply.de

Distributed by:
DENTSPLY Canada
161 Vinyl Court
Woodbridge, Ontario
L4L 4A3 Canada

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Clasificación: ISO 4823

Tipo 3: Consistencia Light-bodied: Aquasil Ultra XLV, Aquasil Ultra LV and Aquasil Ultra Xtra (wash)

Tipo 2: Consistencia Medium-bodied: Aquasil Ultra Monophase, Aquasil Ultra Heavy and Aquasil Ultra Xtra (tray)

Tipo 1: Consistencia Heavy-bodied: Aquasil Ultra Rigid

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DENTSPLY DeTREY GmbH