



Why does everyone love this stuff...



Directions for cleaning of Cartridge Dispensed Aquasil Ultra LV Regular Set Smart Wetting® Impression Material

Dispenser Cleaning Instructions

The dispenser may be cleaned by scrubbing with hot water and soap or detergent. Dispensers or used tips, if left in place for storage, exposed to spatter or spray of body fluids or that may have been touched by contaminated hands, or oral tissues, should be disinfected with a hospital-level disinfectant. Acceptable disinfectants are those that are EPA-registered as tuberculocidal. Sodium hypochlorite (5.25%), isopropyl alcohol and dual or synergized quaternaries are approved disinfectants. Water-based disinfectant solutions are preferred. Some phenolic based agents and iodophor based products may cause surface staining. Agents containing organic solvents, such as alcohol may tend to dissolve the plastic dispenser and plunger. The disinfectant manufacturer's directions should be followed properly for optimum results.

Technique Tip: Unset Aquasil Ultra Impression Material can be removed from clothing with dry cleaning solvent.

DISINFECTION

The impression should be disinfected with a hospital-level disinfectant. Acceptable disinfectants are those that are EPA-registered as tuberculocidal. Iodophors, sodium hypochlorite (5.25%), chlorine dioxide, and dual or synergized quaternaries are approved disinfectants. To disinfect polyvinyl siloxane material, thoroughly soak by spraying or immerse the impression in any recommended hospital level disinfectant except neutral glutaraldehyde for the contact time recommended by the disinfectant manufacturer for optimum results. Water-based disinfectant solutions are preferred.

CASTING AND ELECTROPLATING

The impression should be removed from the disinfectant, rinsed with water and exposed to air to dry for at least the same amount of time it was exposed to the disinfectant solution before it is poured. The impression may be poured up to fourteen days. The material is compatible with a range of die stones. Impressions may be silver or copper-plated